



# Mastatest HiSCC CARTRIDGES

## New High Somatic Cell Count Cartridges

The Mastatest HiSCCtest cartridge identifies the mastitis bacteria infecting cows with high somatic cell counts –within 24 hours.

Knowing the mastitis bacteria causing high SCC and reviewing the cow's mastitis history helps with:

### Protocol 4 easy steps



1 cartridge allows to test 2 cows at the same time to optimize cost and sample management.



1.

Identify cows with a high somatic cell count as it is a good indicator of subclinical mastitis. Mastatest will identify the bacteria causing this infection.



2.

Take a composite milk sample of 4 quarters or from quarter(s) already identified with high SCC. Then pour the sample in the cartridge



3.

Let Mastatest works: Place the cartridge into the lapbox enter the cow number and press the button.



4.

Withing 24h you will be able to make your decision: Treatment or no treatment?

Reporting: S. aureus, CNS, Other Gram+ or Gram- (Note: No antibiotic sensitivity testing)



Use a HiSCC test cartridge only for testing cows with high SCC. For testing clinical mastitis cases use Clinical Cartridges.

Learn more on how easily use Mastatest HiSCC Cartridges



ACHIEVE  
BETTER MILK  
TOGETHER

 **vetoquinol**  
ACHIEVE MORE TOGETHER